

+ ALLEVYN[®] NON-ADHESIVE Foam Dressing is a non-bordered dressing, which is easy to apply and soft and conformable for the patient.^{1,2,3} It is suitable for use with compression therapy and absorbs and retains fluid when compressed.^{4,5}

Smith+Nephew

**ALLEVYN[®]
NON-ADHESIVE**
Foam Dressing

Application



- 1**
- Remove ALLEVYN NON-ADHESIVE Dressing from the pack



- 2**
- Apply the white face to the wound ensuring good contact



- 3**
- Secure with retention dressing e.g. OPSITE[®] FLEXIFIX Transparent Film Roll, OPSITE FLEXIFIX GENTLE Transparent Silicone Adhesive Film Roll, tape or compression bandage

Removal


To remove the ALLEVYN NON-ADHESIVE Dressing, remove the retention dressing or bandage and lift the dressing away from the wound.

Frequency of change

During the early stages of wound management ALLEVYN NON-ADHESIVE Dressing should be inspected according to local clinical protocol. Where the product is used on infected wounds the infection should be inspected and treated as per local clinical protocol. Dressings can be left in place, undisturbed for up to 7 days, or until exudate is visible and approaches to within 1.5cm (⁵/₈in) from the edge of the dressing, whichever is sooner.

The diagram below is for guidance only.

The decision of when to change ALLEVYN NON-ADHESIVE Dressing should be dependent upon clinical assessment and local protocols should also be taken into consideration.

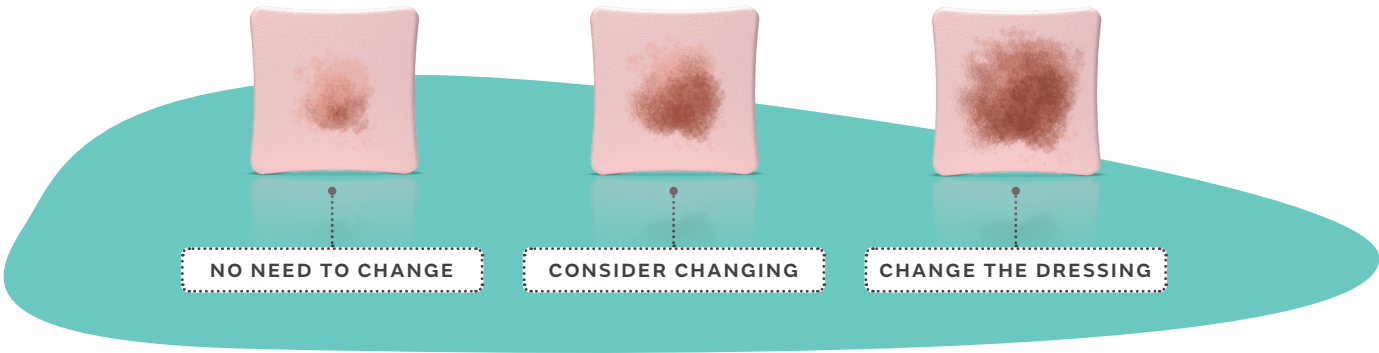

ALLEVYN NON-ADHESIVE Foam Dressing ordering codes

Size	Code	PIP Code	NHSSC Code	Carton	Size	Code	PIP Code	NHSSC Code	Carton
5cm x 5cm / 2in x 2in	66007643	207-0787	ELA129	10	20cm x 20cm / 8in x 8in	66007638	207-0837	ELA133	10
10cm x 10cm / 4in x 4in	66007637	207-0811	ELA131	10	40cm x 70cm / 15 ³ / ₄ in x 27 ¹ / ₂ in	66000663	275-9983	ELA1075	2
10cm x 20cm / 4in x 8in	66007335	207-0829	ELA101	10					

Smith+Nephew
Croxley Park, Building 5
Hatters Lane, Watford
Hertfordshire WD18 8YE

T +44 (0) 1923 477100
F +44 (0) 1923 477101

www.smith-nephew.com/allevyn

◇Trademark of Smith+Nephew
All Trademarks acknowledged
© 2024 Smith+Nephew
43749-uki 07/24

For detailed product information, including indications for use, contraindications, precautions and warnings, please consult the product's applicable Instructions for Use (IFU) prior to use.

References

1. Foster AVM, Greenhill MT, Edmonds ME. Comparing two dressings in the treatment of diabetic foot ulcers. *Journal of Wound Care*. 1994;3(5):224–228. **2.** Kurring PA, Roberts CD, Quinlan D. Evaluation of a hydrocellular dressing in the management of exuding wounds in the community. *British Journal of Nursing* 1994;3(20):1049–1050, 1052–1043. **3.** Smith +Nephew 2019. Volunteer evaluation of AWM product features. Internal report. EO.AWM.PCSgen.007.v1. **4.** Smith+Nephew 2018. Project Peridot Testing - ALLEVYN NON-ADHESIVE Dressings. Internal report. DS/18/264/R. **5.** Franks PJ, Moody M, Moffatt CJ, et al. Randomised trial of two foam dressings in the management of chronic venous ulceration. *Wound Repair and Regeneration*. 2007;15(2):197–202.