

clinell®


Easyclean Commode

Commodes and over the toilet equipment must be cleaned with a proven sporicidal agent after every patient, as per company policy.

It is important to always wash and dry hands, and wear the recommended PPE when cleaning, using the following procedure.



Instructions for use

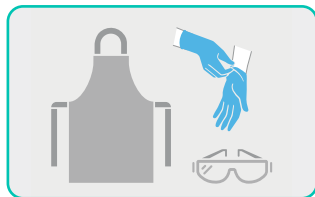

Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol).



Remove the wipe from the packet and activate it by wetting under a tap or bucket. Should your wipe dry out or become soiled, replace it with a new wipe.




Once activated, squeeze into a sink or bucket to remove any excess water.



Remove the seat cover and wipe the top in an 'S' shaped pattern. Repeat on the underside and leave to air dry, allowing sufficient contact time with the disinfectant.




Unclip and remove the backrest. Wipe the front and back of the rest in an 'S' shaped pattern from top to bottom. Leave to air dry on a clean, dry surface.

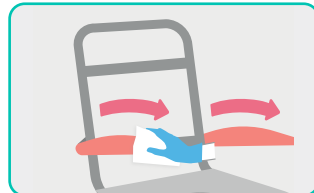

Remove and wipe arms. Leave to air dry on a clean, dry surface.




Remove seat and wipe in an 'S' shaped motion. Repeat for the underside and leave to air dry on a clean, dry surface.

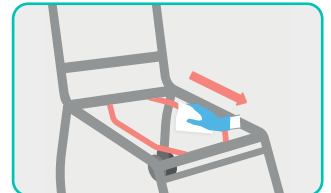


Remove the bedpan rack and carefully clean both sides. Leave to air dry on a clean, dry surface.




Clean the remaining frame from top to bottom. Then, unclip the footrests and clean thoroughly. Leaving both to air dry on a clean, dry surface.

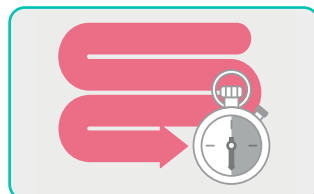

Wipe the underside of the wheels, leaving to air dry and then leave all parts to air dry to allow the disinfectant enough contact time to work.

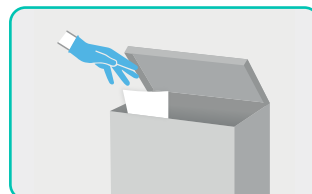

Dispose of cleaning wipes as per local protocol.

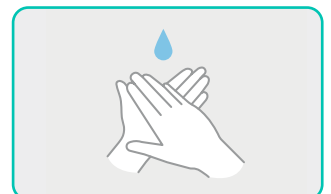

Remove PPE, wash or decontaminate hands and if required complete a record of cleaning. When fully dry, carefully reassemble the commode.

JBN220582