

Disgard Sharps Pad User Guide

For the safe collection and disposal of
used blades and needles.

The sequences shown here should be
used as guidance. Please refer to local
hospital guidelines for further information.



01



Always remember to wash and disinfect
hands and wear the appropriate gloves
before performing any procedure.

02



The pad case can be secured to a surface, such
as the patient's drape by removing the small
patch on the base to reveal an adhesive pad.

03



Inside, remove the paper strip covering to
reveal a yellow foam pad with an adhesive
surface.

04



The pad case is now ready to receive sharps
– be extremely careful when removing
any sharp object from its original source.

05



Place any discarded sharps onto the yellow
foam area of the pad using the appropriate
instrument.

06



When the procedure is complete, fold the lid
over at the hinge and seal.

07



The transparent plastic case allows the
sharps to remain visible for counting.

08



Dispose of the complete unit in the nearest
sharps bin provided.



For more information:

Tel: 01530 278 111 | Email: info@365healthcare.com | www.365healthcare.com