

Fitting Your Respirator User Guide

UniProtect™ particulate respirators are recommended for use where protection of the respiratory system is essential.

Properly donning your respirator may feel a little awkward at first, but it will become easier with repeated applications.

Do not proceed with your activities until you have checked your respirator fit is successful.

When using a respirator, the blue or white nose band side MUST be worn facing outward and upward in order to provide the best possible fit.

The sequences shown here should be used as guidance. Please refer to local hospital guidelines for further information.

Fit Testing & Checking

Fit check your respirator every time you wear it. Forcefully inhale and exhale several times. The respirator should collapse slightly when you inhale and expand when you exhale. You should not feel any air leaking between your face and the respirator. If the respirator does not collapse and expand OR if air is leaking out between your face and the respirator, then you have not achieved a good facial fit. Adjust your respirator until the leakage is corrected and you are able to successfully fit your respirator.

Important: If after trying these tips you are still unable to successfully fit check your respirator, please contact us.



01



Separate the mask to open fully.

02



Gently pre-bend the nose piece to conform the mask to your face.

03



Hold the mask upside down to expose the two headbands.

04



Separate the headbands using your index and middle fingers.

05



Tip your head slightly forward and place the base of the mask firmly under your chin.

06



Position the mask on your face and bring the headbands over your head.

07



Place the first headband at the nape of your neck and the remaining headband at the crown of your head.

08



Form the nose piece tightly across the bridge of your nose and face. Adjust the mask to achieve a good facial seal.



365
HEALTHCARE

For more information:

Tel: 01530 278 111 | Email: info@365healthcare.com | www.365healthcare.com