

Sharps Safety Station Box

User Guide

For the safe collection, storage and disposal of used blades, suture needles and other sharps. The sequences shown here should be used as guidance. Please refer to local hospital protocol.

01



Open the box and remove the paper strip on the inside of the lid to reveal a foam pad with an adhesive surface. Used sharps can be placed onto the foam area.

02



Used sharps can be put (1) on the adhesive foam pad, (2) deposited in the clear container box or (3) accounted for within the numbered main section to ensure safe collection, storage and disposal.

03



When the procedure is complete, fold the lid over at the hinge and seal the box using the locking system. Dispose of the locked box in the main sharps bin.