

clinell® Universal Spray

Kills more than 99.99% of germs

Clinell Universal Spray can be used for regular surface disinfection and cleaning and disinfection of non-invasive medical devices.

Please dispose of bottle once empty in the waste bin provided.



CDS500 - Universal Spray 500ml

The following equipment should be routinely decontaminated with Clinell Universal Spray as per local protocol / policy.

(Examples of use - this is not an exhaustive list)

- | | | |
|--------------------------|---------------------------|------------------------|
| - Ultrasound transducers | - ECG machines | - Blood pressure cuffs |
| - Pulse oximeter probes | - Incubators | - Monitors |
| - Doppler probes | - Blood pressure machines | - Mattresses |
| - Stethoscope | - Radiology equipment | |

Instructions for use




Wash or decontaminate your hands and put on PPE (risk assessment to ensure appropriate PPE is worn, as per local protocol).



Turn nozzle and spray surface.



Working from top to bottom, clean to dirty, wipe in an 'S' shaped pattern.

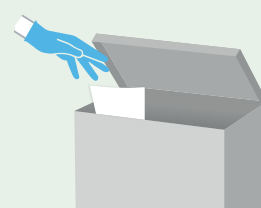

Should your wipe dry out or become soiled, replace it with a new wipe. Let the surface air dry to maximise contact time.

Please ensure that you follow the manufacturer's guidelines.